

INSTRUCTIONAL IMAGES



STEP 1:
Insert open tube
end through the
middle vent hole.



STEP 2:
Connect open tube
end with
bubbler cap.



STEP 3:
Attach bubbler to
your desired base
position using the
suction cup.



STEP 4:
Place bath over
open toilet (seat up)
and push the basin
in a downwards
direction.



STEP 4:
When pushed down
the basin pops out
to extend to it's full
capacity.



STEP 5:
Add warm water
into the basin,
ensuring the
perineum is
submerged in water.



STEP 6:
Sit over the Sitz
bath and depress
the bubbler to start
rinsing. Enjoy for
15-20 minutes.



STEP 8:
After use, empty
contents directly
into the toilet and
flush.



STEP 8:
Thoroughly clean
the bath and hang
for next use.